

# HUGE

## LuFill HS

High-Strength Universal  
Glass Ionomer Material



Approved by FDA  
EU MDR Certification

FDA  
Approved



## New Color System!

LuFill HS comes in **Vita Color** system. The color of the material is more consistent with the color of the teeth and has excellent aesthetic effects.

A2

A3

A3.5



## Product Overview

LuFill HS is VinciSmile's new upgraded product. It is an excellent self-adhesive, bulk glass ionomer cement that guarantees reliable and lasting restorations. Due to its high strength, this product performs well when used in routine clinical restorations.

## Simple Operation and Excellent Performance

- Ideal particle size, easy mixing and good operating experience
- **High compressive (>200MPa) and flexural strength** assure durability, longevity, and integrity
- Durable physical properties, high acid resistance and low oral solubility
- Continuous fluoride ion release helps to prevent secondary dental cavities
- X-ray resistance



# Clinical Operation



1. Prepare the tooth



2. Apply the conditioner



3. Take powder and liquid, mixing ratio 1:1



4. Mix the powder and liquid



5. Fill and trim



6. Waterproof treatment



7. Finish



## Order Details

*Direct restorative treatment for efficient esthetics*

### LuFill HS

Standard: 30g Powder+25g Liquid


Mini: 10g Powder+10g Liquid

colour: A2  A3  A3.5 

# Technical Parameters

Technical Data	LuFill HS	
	For Luting	For Filling
Working time	2min10s	3min00s
Net setting time	3min15s	2min45s
Film thickness	13µm	/
Compressive strength	/	>200Mpa

\* All data from HUGE laboratory

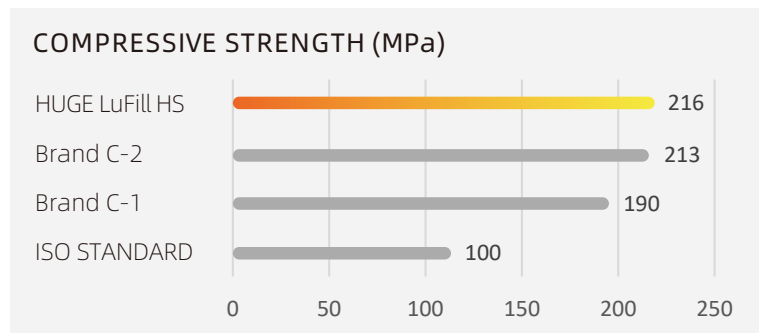

## Shandong Huge Dental Material Corporation

Add /No.68 Shanhai Road, Donggang District, Rizhao City, Shandong Province, 276800, P.R.China.  
Tel/ +86 (633)2277268 marketing@hugedental.com www.hugedental.com

Facebook Huge Dental Instagram Huge Dental Youtube Huge Dental